


User Instruction Manual

Oxford® Full Body Sling (Clips)

To avoid injury, read user manual prior to use.

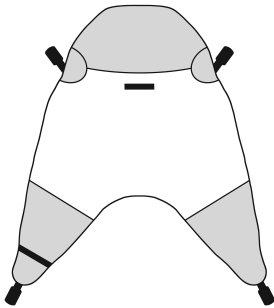

FOR USE WITH PASSIVE HOISTS ONLY.

All drawings are for illustrative purposes only.

SPECIAL SLING ORDERS:

On occasions, material, dimensional and other changes outside of the standard specification are requested. Please be advised, that aside of any model specific references, fitting, washing and safety guidelines remain applicable. If you are in any doubt, please contact your authorised Oxford service agent or Joerns Healthcare directly.


⚠ WARNING

- **OXFORD RECOMMENDS THE USE OF GENUINE OXFORD PARTS.** Oxford sling and lift products are designed to be compatible with one another. For country specific guidance on sling use and compatibility, please refer to the sling label or contact your local market distributor or Joerns Healthcare.
- For the safety of the patient and carer; before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.
- **CHECK** sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.
- **DO NOT** alter slings. Destroy and discard worn slings.
- **NEVER** leave a patient unattended.
- **DO NOT** exceed the rated capacity of the sling or lift.
- **DO NOT** attempt to re-position a patient by pulling on the sling loops.

Introduction

The Oxford Full Body Sling (Clips) is a general-purpose sling with integrated head support and padded legs, offering excellent comfort and support for clients. Quick and easy to apply, this sling utilises clip attachment points for use on passive hoists with a 4-point cradle.

Fitting the Sling from a Seated Position

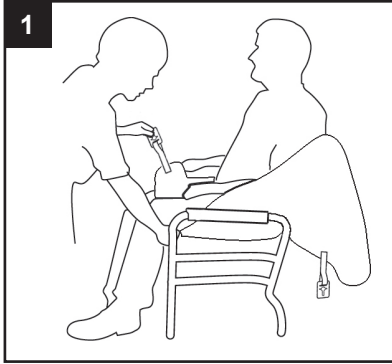


The sling is fitted to the patient in the traditional way. Fitting from the seated position requires the carer to raise the clients leg and feed the leg support piece under and up between the legs. Repeat this procedure for other leg.

Ensure the sling is not twisted or creased under the thigh. Bring both leg pieces together and feed the right leg through the modesty strap located on the opposite leg.


Position the rear section of the sling up and around the shoulders and head.

Fitting the Sling from a Lying Position

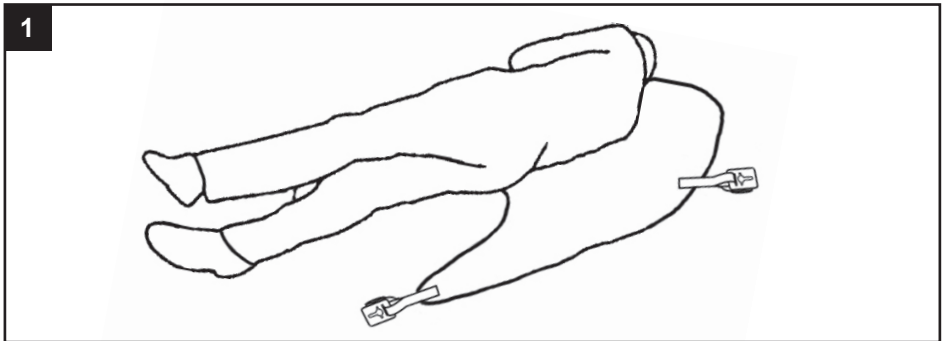


Draw sheet roll the client onto the sling, ensuring that the top of the commode aperture is at the base of the spine. Carefully feed the leg sections under and up between the patient's legs, ensuring that the sling is positioned comfortably and not twisted or creased under the thighs. Configure the leg straps in the same manner as with the seated position above.

Fitting the Sling to the 4-Point Cradle

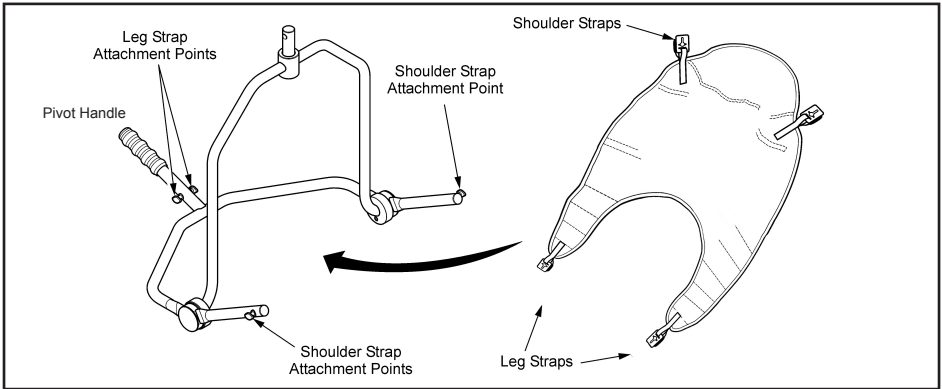

When fitted to the patient, the sling is connected to the 4-point cradle by using the Securi3 clip system (step by step guide as follows). To remove, simply reverse this procedure.

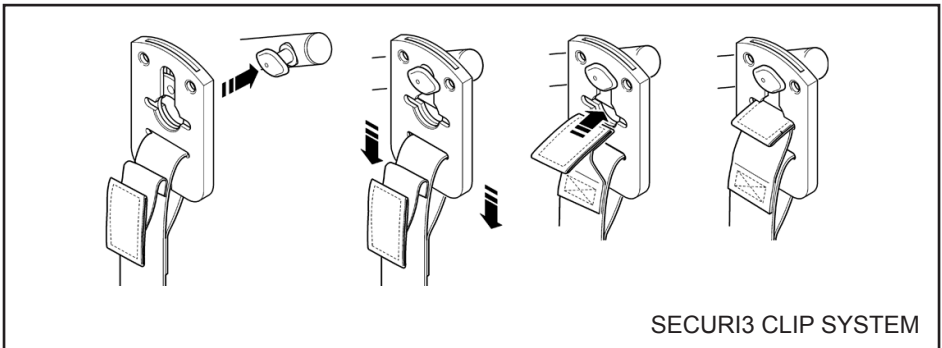

⚠ WARNING - SECURI3 CLIP SAFETY

Ensure all green safety loops are visible before lifting.

Once the sling is securely connected to the 4-point cradle, the client is ready for lifting.

NOTE: The client's position can be altered by pushing down on the 4-point cradle pivot handle (see above).

Technical Specifications

STANDARD	BS EN ISO 10535
SAFE WORKING LOAD	SIZE DEPENDENT (SEE BELOW)

Sizing

SIZE	SWL
XS (BROWN)	227kg / 500lbs
SMALL (RED)	227kg / 500lbs
MEDIUM (YELLOW)	227kg / 500lbs

SIZE	SWL
LARGE (GREEN)	227kg / 500lbs
XL (BLUE)	284kg / 600lbs

Special sizes can be made to order. Please contact your authorised Oxford distributor or Joerns Healthcare for more information.

Washing Instructions



Machine wash at 85°C.



DO NOT wash with bleach. Bleach will damage the sling's material and make it unsafe for use.



Cool tumble dry, air dry or dry at very low temperature.



DO NOT dry clean.

⚠ WARNING

Slings can suffer damage during washing and drying and should be checked carefully before each use.

PLEASE NOTE: Additional slings are available to meet individual patient needs. We advise that you always seek the advice of a trained clinician, authorised Oxford distributor or Joerns Healthcare prior to purchase or use.